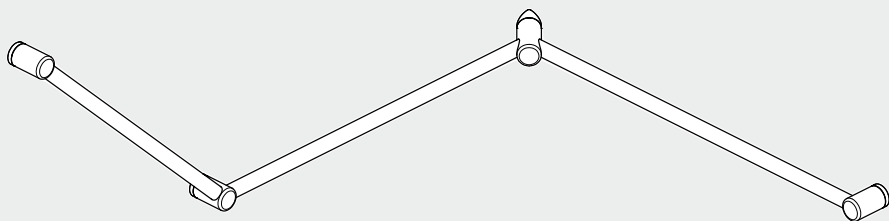


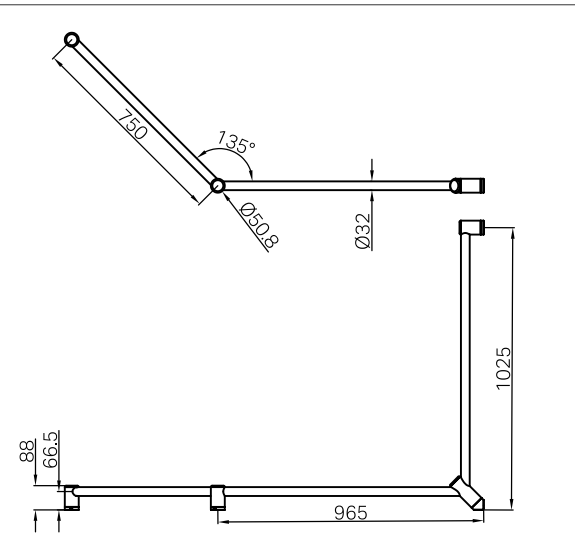
Mecca Care 32mm DDA Accessible Toilet Grab Rail Set 45 Degree Continuous

Nero



Versions:202503

Drawing









Warning:

These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard.

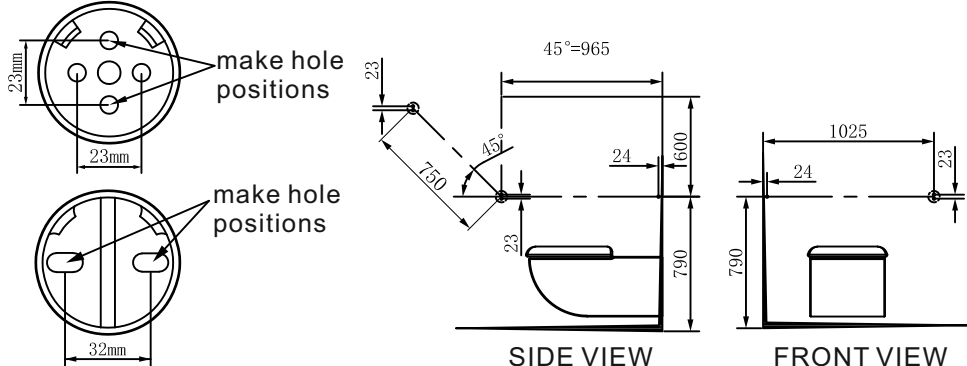
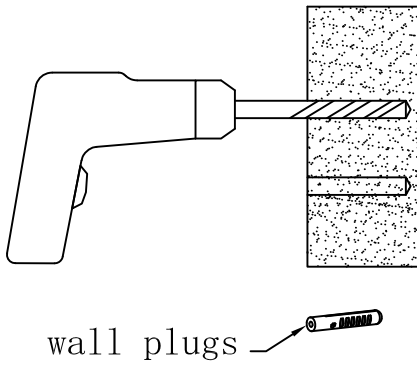
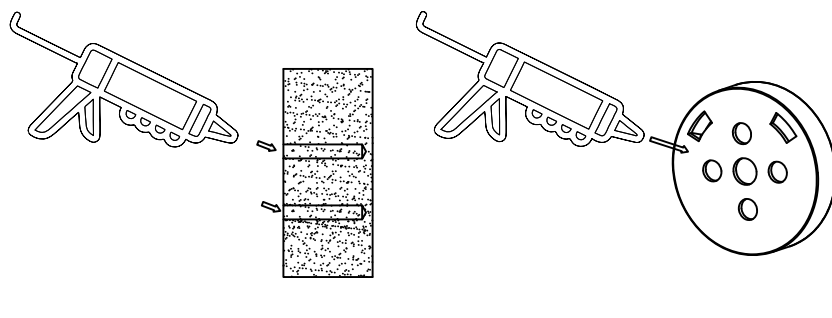
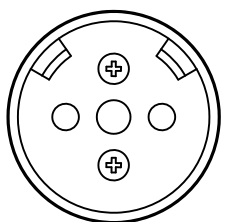
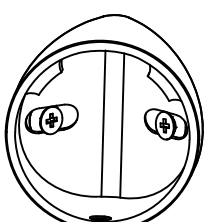
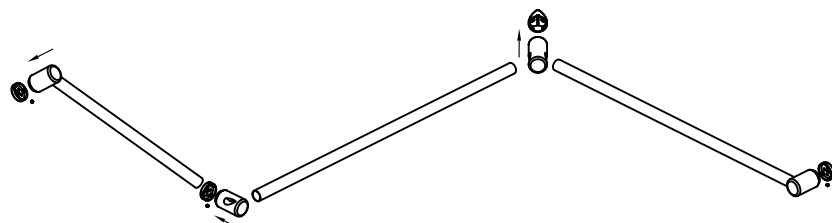
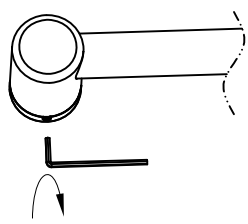
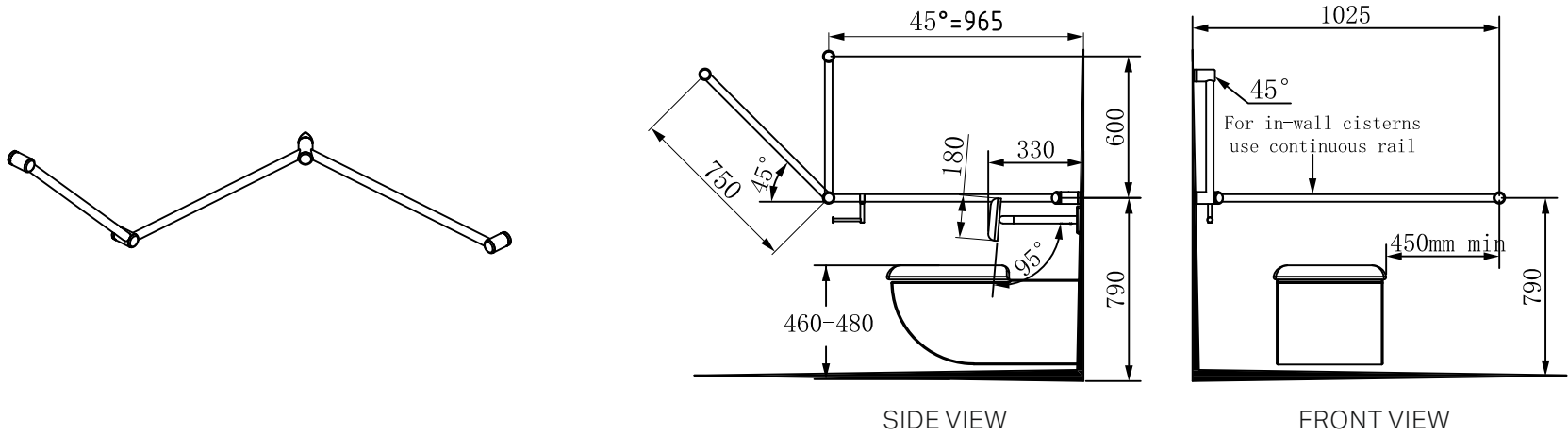
It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the

Finish & SKU

 Chrome NRCR3245cCH	 Gun Metal NRCR3245cGM	 Brushed Bronze NRCR3245cBZ
 Matte Black NRCR3245cMB	 Brushed Gold NRCR3245cBG	 Brushed Nickel NRCR3245cBN

*Dimensions are nominal measurements only.

INSTALLATION INSTRUCTIONS

 <p>make hole positions</p> <p>make hole positions</p> <p>23mm</p> <p>23mm</p> <p>32mm</p> <p>23</p> <p>45°=965</p> <p>24</p> <p>1600</p> <p>790</p> <p>1025</p> <p>24</p> <p>83</p> <p>SIDE VIEW</p> <p>FRONT VIEW</p>	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall refer to left picture. (minimum 2 supplied screws per bracket).</p> <p>Tip: Rail height is 790mm above the ground.</p>
 <p>wall plugs</p>	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 3mm hole for screws into timber or 8mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
<p>WALL BRACKET</p>  <p>CORNER BRACKET</p> 	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Push sliders onto wall brackets, refer to left picture.</p> <p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p>
	<p>6. Tighten grub screws with allen key.</p> <p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p>
 <p>7. Finished.</p> <p>45°=965</p> <p>1025</p> <p>45°</p> <p>For in-wall cisterns use continuous rail</p> <p>750</p> <p>180</p> <p>330</p> <p>600</p> <p>790</p> <p>460-480</p> <p>95°</p> <p>450mm min</p> <p>SIDE VIEW</p> <p>FRONT VIEW</p> <p>Note: The grab rail can be mounted on either left or right hand side.</p>	

Nero



MECCA CARE
DDA CARE 32MM



Warranty Policy