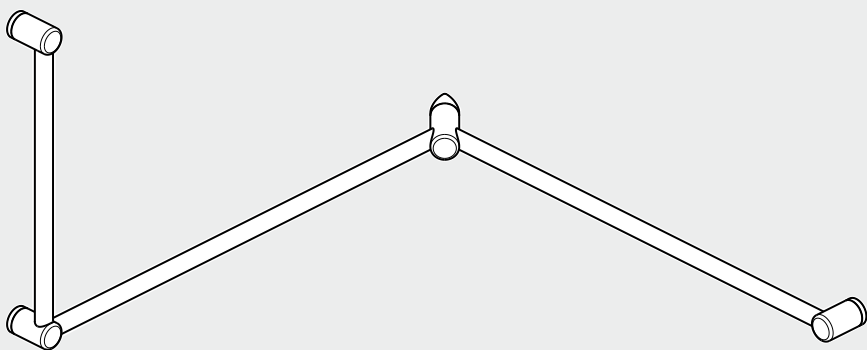


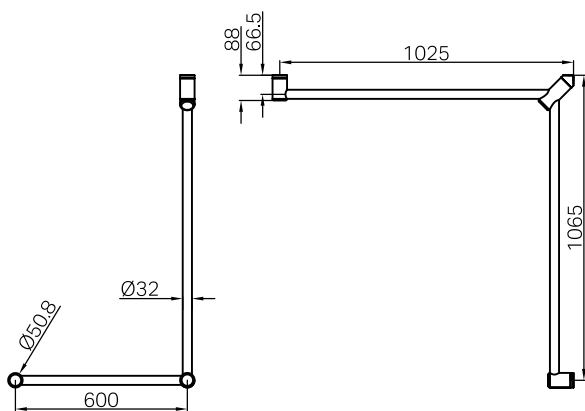
Mecca Care 32mm DDA Accessible Toilet Grab Rail Set 90 Degree Continuous

Nero



Versions:202506

Drawing



Warning:

These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard. It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the

Finish & SKU

Chrome
NRCR3290cCH

Gun Metal
NRCR3290cGM

Brushed Bronze
NRCR3290cBZ

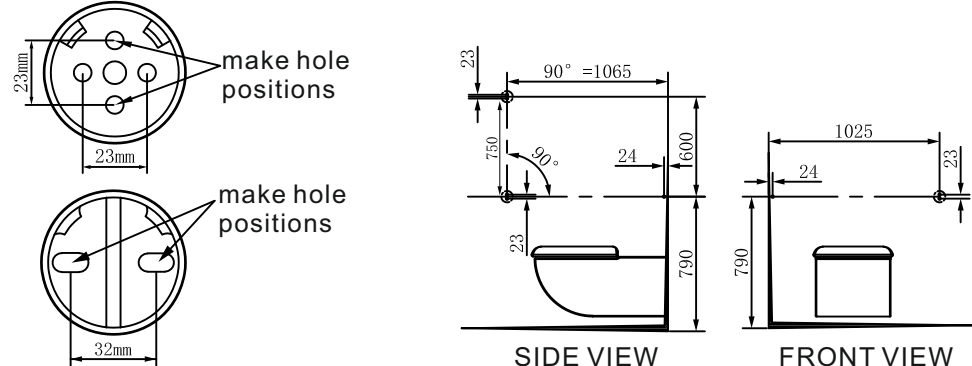
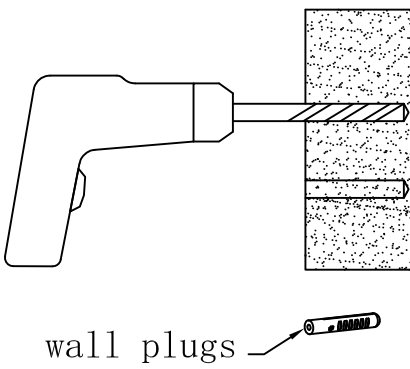
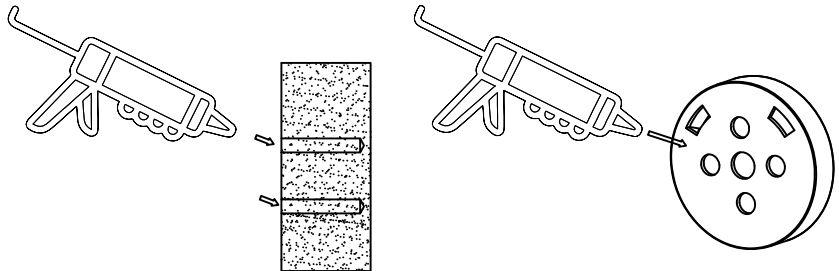
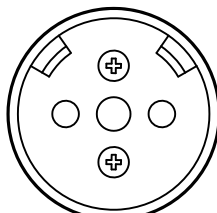
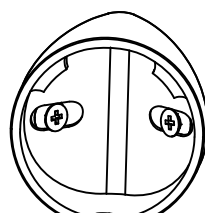
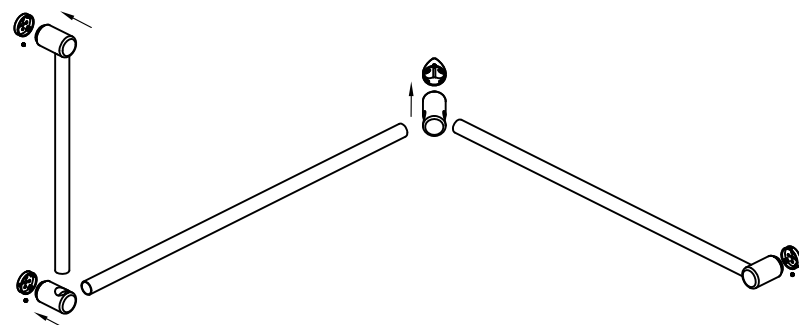
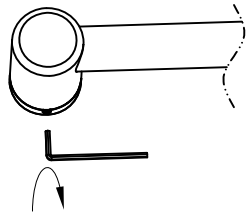
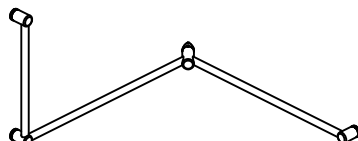
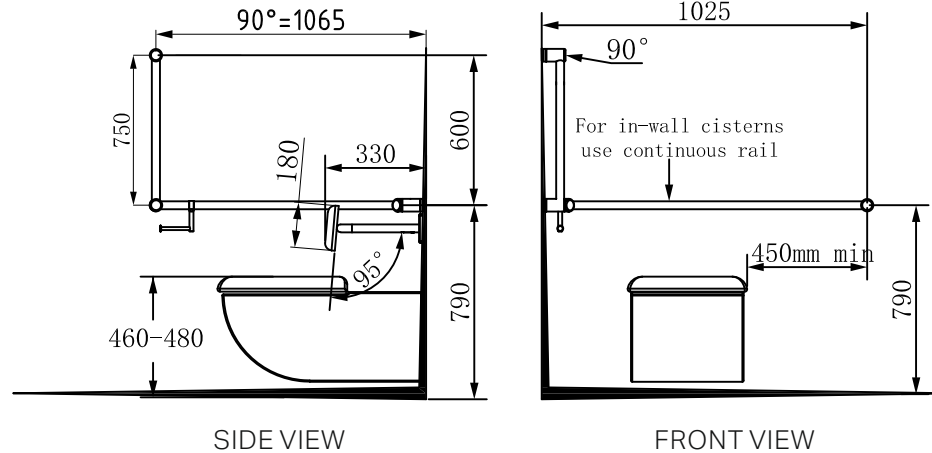
Matte Black
NRCR3290cMB

Brushed Gold
NRCR3290cBG

Brushed Nickel
NRCR3290cBN

*Dimensions are nominal measurements only.

INSTALLATION INSTRUCTIONS

 <p>make hole positions</p> <p>make hole positions</p> <p>SIDE VIEW</p> <p>FRONT VIEW</p>	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall refer to left picture. (minimum 2 supplied screws per bracket).</p> <p>Tip: Rail height is 790mm above the ground.</p>
 <p>wall plugs</p>	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 3mm hole for screws into timber or 8mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
<p>WALL BRACKET</p>  <p>CORNER BRACKET</p> 	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Push sliders onto wall brackets, refer to left picture.</p> <p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p>
	<p>6. Tighten grub screws with allen key.</p> <p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p>
 <p>90 Degree Continuous</p>	<p>7. Finished.</p>  <p>SIDE VIEW</p> <p>FRONT VIEW</p> <p>Note: The grab rail can be mounted on either left or right hand side.</p>

Nero



MECCA CARE
DDA CARE 32MM

