


# Mecca Care 25mm Grab Rail And Adjustable Shower Rail Set 900mm


Nero

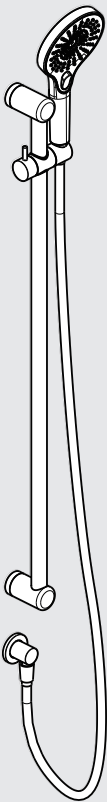

Versions:202506



## Drawing

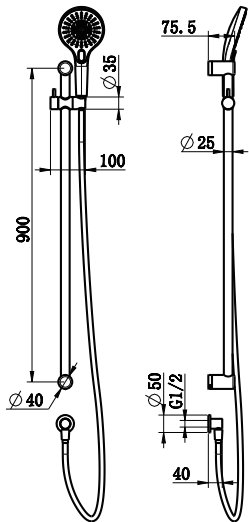

## Specifications

**SKU**  
NRCS0004

### Warning:

These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard. It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the rail which can result in injury to user and damage to the wall.


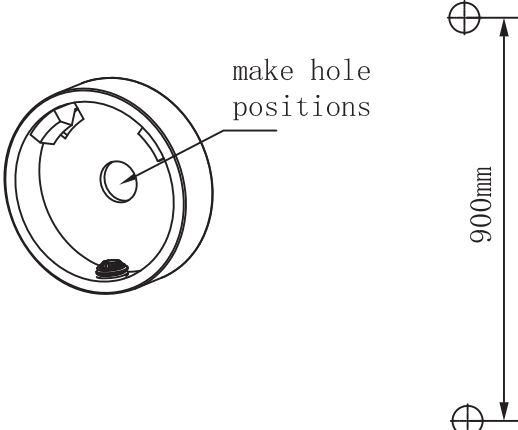
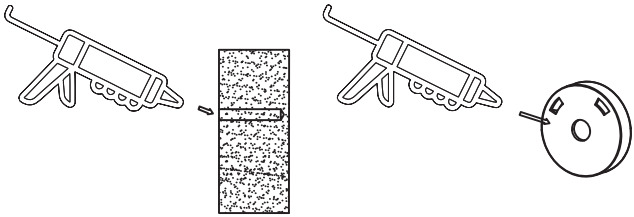
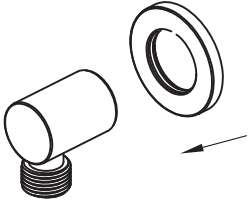
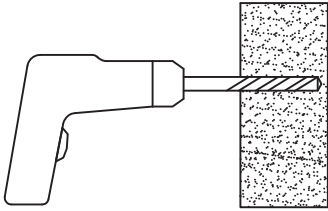
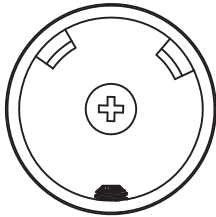
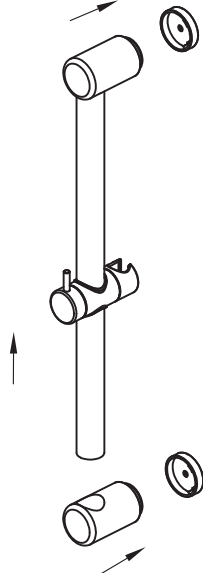
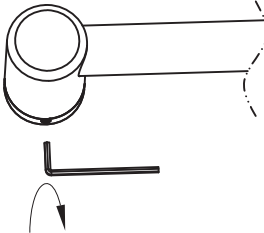
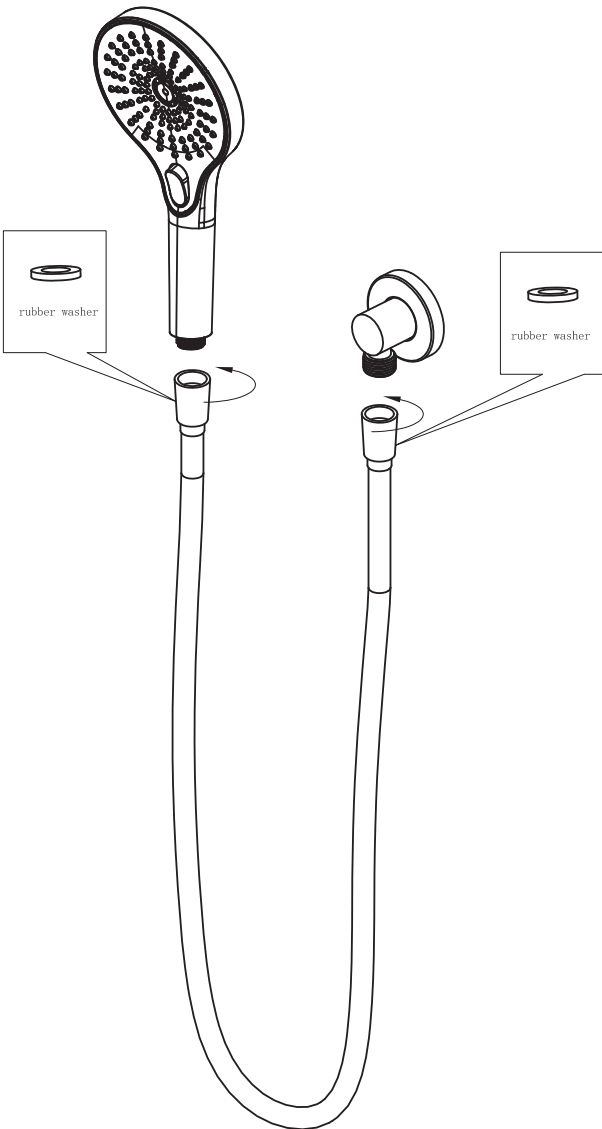
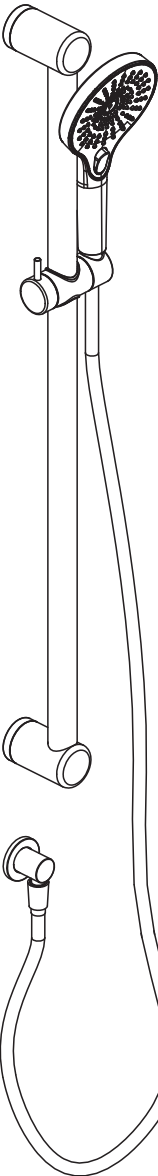
### Finish

 <b>Chrome</b> NRCS0004CH	 <b>Gun Metal</b> NRCS0004GM	 <b>Brushed Bronze</b> NRCS0004BZ
 <b>Matte Black</b> NRCS0004MB	 <b>Brushed Gold</b> NRCS0004BG	 <b>Brushed Nickel</b> NRCS0004BN

## Features

1. Australian owned and designed.
2. 1.5m PVC hose.
3. The rail can be shortened during installation to suit your bathroom.
4. 25mm diameter rails are suitable for heavy duty towel rails and smaller hands, in non-DDA bathrooms.
5. Gun metal, brushed gold and brushed bronze colours are PVD coated for long term durability.
6. WELS rating is 3 STAR 9L/MIN.
7. WATERMARK license is WM-060093.

\*Dimensions are nominal measurements only.

<div data-bbox="327 290 638 388"><p>Use Teflon Tape</p></div> <div data-bbox="174 507 632 537"><p>1. Put teflon tape onto outlet thread.</p></div>	<div data-bbox="800 246 1283 647"><p>make hole positions</p><p>900mm</p></div> <div data-bbox="758 655 1354 804"><p>4. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall.</p></div>	<div data-bbox="1373 314 1965 516"></div> <div data-bbox="1367 655 1976 715"><p>6. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p></div>
<div data-bbox="291 647 522 834"></div> <div data-bbox="195 923 512 952"><p>2. Slide plate onto elbow.</p></div>	<div data-bbox="789 854 1094 1050"></div> <div data-bbox="758 1080 1066 1124"><p>wall plugs</p></div> <div data-bbox="758 1258 1346 1442"><p>5. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p><p>Tip: 4mm hole for screws into timber or 10mm hole for wall plugs.</p></div>	<div data-bbox="1535 937 1736 1139"></div> <div data-bbox="1383 1258 1965 1442"><p>7. Install mounting brackets by screwing onto wall.</p><p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p></div>
<div data-bbox="285 1486 474 2021"></div> <div data-bbox="161 2050 737 2294"><p>8. Slide the shower slider onto rail, then slide rail tubes into end brackets and push onto wall brackets, refer to above picture.</p><p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p></div> <div data-bbox="249 2332 495 2549"></div> <div data-bbox="165 2623 644 2742"><p>9. Tighten grub screws with allen key.</p><p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p></div>	<div data-bbox="842 1516 1404 2570"></div> <div data-bbox="758 2623 1438 2718"><p>10. Connect hand shower and elbow by shower hose.</p><p>Tip: There are two rubber washer in the shower hose.</p></div>	<div data-bbox="1646 1546 1793 2644"></div> <div data-bbox="1518 2718 1913 2748"><p>11. Test and check for leaks.</p></div>

# Nero



MECCA CARE  
ASSISTED LIVING 25MM

